



Juno™ Portable Handheld  
Video Magnifier with  
OCR and Touch  
**Quick Start**



AMERICAN  
PRINTING  
HOUSE




*All Rights Reserved.*

[www.APH.ORG](http://www.APH.ORG)

Catalog number 1-00380-00

UDC-02332 Rev. A

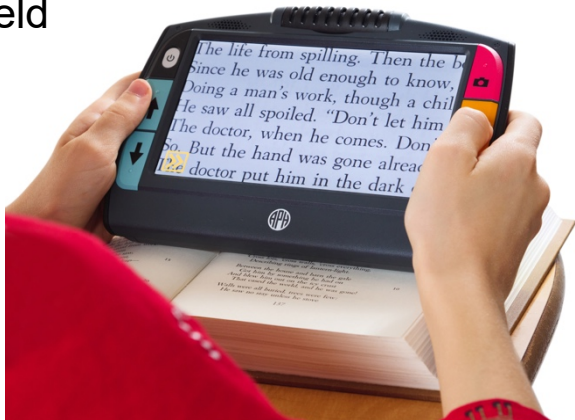


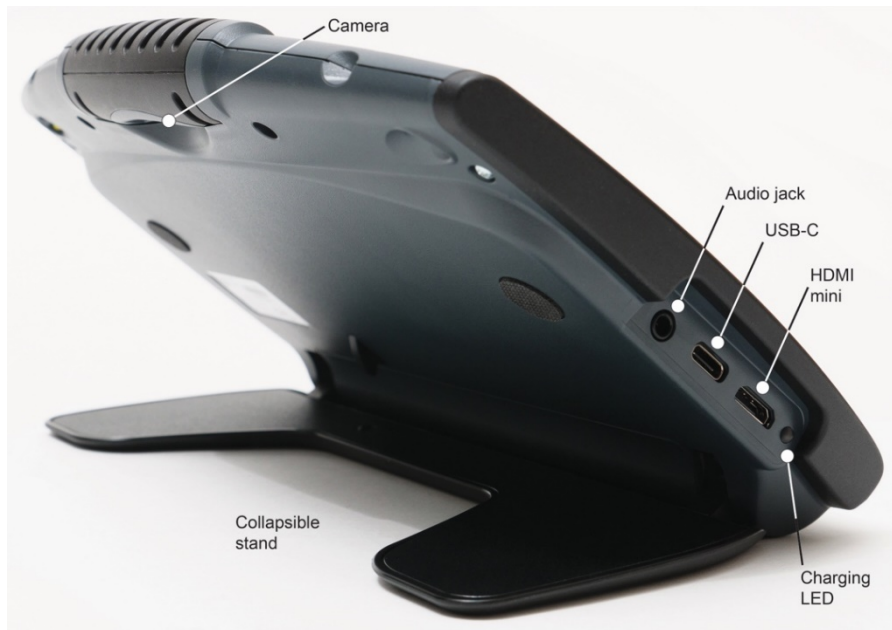
## Charging Juno

Begin charging Juno before first use. When the light on the left side of the device is red, the battery needs to be charged. You can use the device for about three hours once the battery is fully charged. The charging LED light will turn green when the battery is charged.

To charge Juno:


1. Insert the USB Type C cable into the USB-C slot on the left side of the device.
2. Attach the power adapter to the USB-C cable, and plug the power adapter into an electrical outlet. You can alternately insert the cable into a computer without using the adapter, but this charging method is slower.

3. Power on Juno by pressing the white Power button . You can work when the LED is solid red, which means that the battery is charging. However, you should leave it plugged in until the light turns green. The image on the following page is a view of the back and left sides of the device.
4. Begin using Juno. Go to the next section, “First Use.”



## First Use

The first time you use Juno, you must follow these steps. You cannot exit once you start these steps and cannot use the device until you complete them. However, you can change these settings later if needed.

1. Plug in and power on Juno. See the previous section, “Charging Juno.”
2. On the **Select Juno Voice**, tap to select a female or male voice to narrate as you navigate Juno. Only English is available.
3. Tap the **Next** button . The **Select Date** screen is displayed.




4. Swipe up or down in each column and tap to select the month, day, and year. Tap the

**Previous** button  to return to the previous screen.

5. Tap the **Next** button . The **Select Time** screen is displayed.

6. Swipe up or down in each column and tap to select hours, minutes, and AM or PM.



7. Tap the **Next** button  to complete the setup. The Live view is displayed. Refer to the *Juno User Guide* for step-by-step instructions.







## Hardware Buttons

See the *Juno User Guide* for more information about the hardware buttons.

Button	Name	Color	Location	Description
	<b>Power</b>	White	Top left	Press to power on the device. Short press to put on standby. Long press to power off.
	<b>Color Mode</b>	Blue	Middle and lower left	Press to cycle through true color, grayscale, and 22 high contrast color combinations.



Button	Name	Color	Location	Description
	<b>Capture</b>	Red	Top right	Press to capture a snapshot or go to live view. Optionally save a capture, recognize text, and read or listen to the text.
	<b>Zoom in/out</b>	Yellow	Middle and lower right	Press and hold for continuous zoom, press and release to incrementally zoom, or pinch and zoom on touch screen to increase or decrease magnification.

## Button Bars on the Screen



Tap the **Restore** button in the lower left corner of the screen to display the button bars on the left and right sides of the screen.



Tap the **Hide** button to no longer display the button bars.



Main Menu



LED Lights  
On and Off Toggle



Volume and  
Speech Rate



Lines & Masks  
Toggle



Restore & Hide  
Button Bars



Focus Unlocked &  
Locked Toggle

## Main Menu Structure



See the *Juno User Guide* for features associated with these menus.





## Files Menu

---



File Information screen



Edit screen



Record Audio Tag screen



Transfer screen



## About Menu

---



Support screen

Support

Acknowledgements screen

Acknowledgements

Juno User Guide

Manual

## Brightness Menu

---



## Settings Menu

---



The **Settings** menu has four sub-menus, as shown on the following pages:

- Reading Settings
- Menu Settings
- User Settings
- System Settings

## Reading Settings Menu

---



Viewing Modes screen



**Default viewing mode (top right button)**

Image mode



Teleprompter mode



Ticker Tape mode





Viewing Modes screen (continued)



**Default highlighting mode (bottom right button)**

Boxed



Underlined



Select Reading Language screen



## Menu Settings Menu

---



Talking Menu screen



Voice Selection screen



Color Theme screen



Font Size screen



## User Settings Menu

---



Color Modes screen



Standby Time screen



Juno Storage screen



## System Settings Menu

---



Time & Date screen



Default Save Types screen



Update Juno screen



Reset All Settings screen

