




Neptune

Quick Start Guide

Keypad Controls

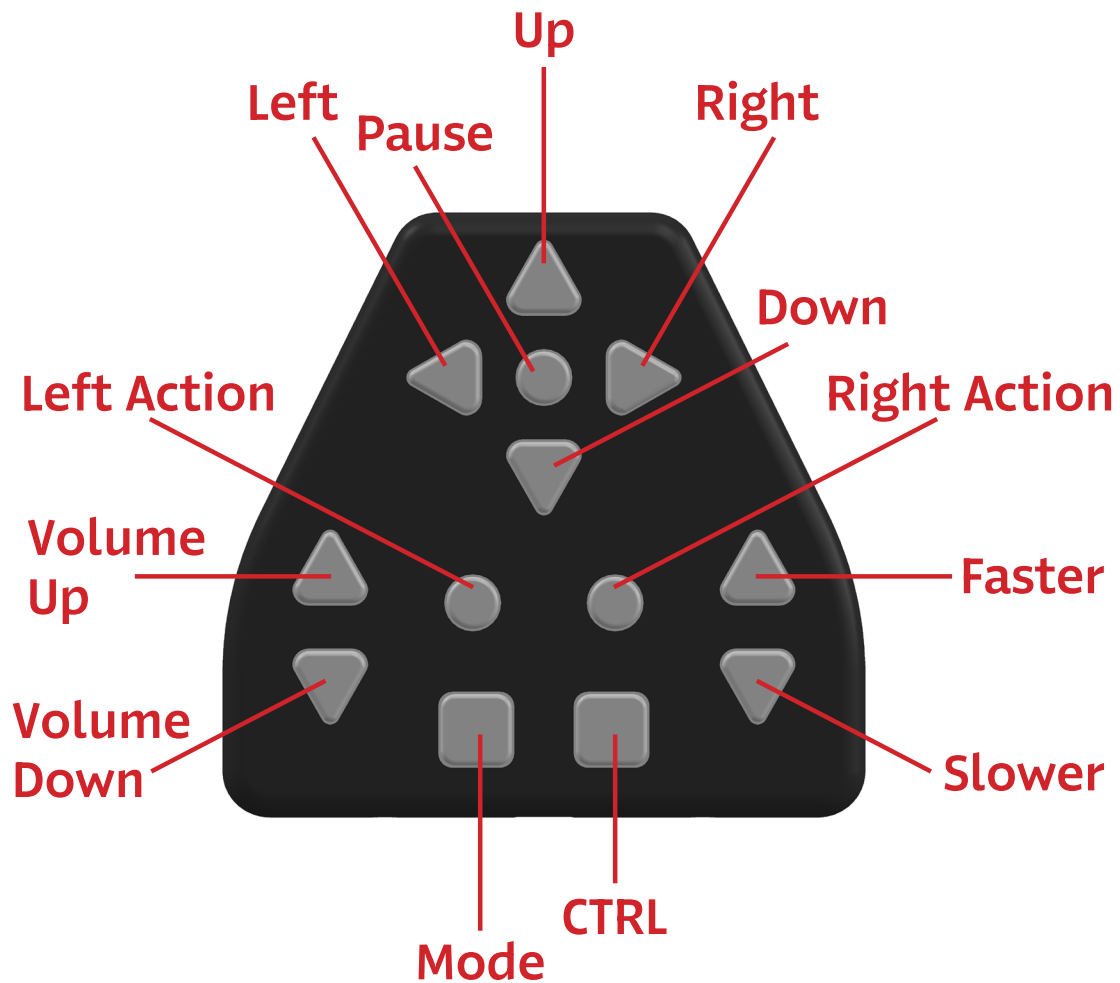

Pause/resume reading	Pause
Move forward/backward	Left/Right Arrow
Forward/backward granularity (character, word, sentence, or paragraph)	Up/Down Arrow
Increase/decrease font size	CTRL + Volume Up/Down
Change text colors	CTRL + Faster/Slower Speed
Enable/disable auto capture	Mode + Right Action
Take a snapshot	Right Action
Change voice/language	CTRL + Left Action
Enter menu	CTRL + Mode
Switch from speaker to Bluetooth	Mode + Left Action
Column mode	CTRL + Right Action

Getting Started with Neptune

Thank you for purchasing the Neptune, a portable Optical Character Recognition device that scans text and instantly begins reading it out loud. Neptune comes with a detachable screen that displays and highlights text in customizable font sizes and colors as it reads. On the next few pages, you will find instructions to set up and begin using your Neptune.

Replacement Parts:

13-inch Monitor: 1-00431-00

AC Power Supply/Charger: 1-00432-00


Keypad: 1-00433-00

Follow this guide to get started with Neptune


1. Press the Open/Close button and lift to separate Neptune from the baseplate.




2. Lift the Neptune body until it clicks into place.

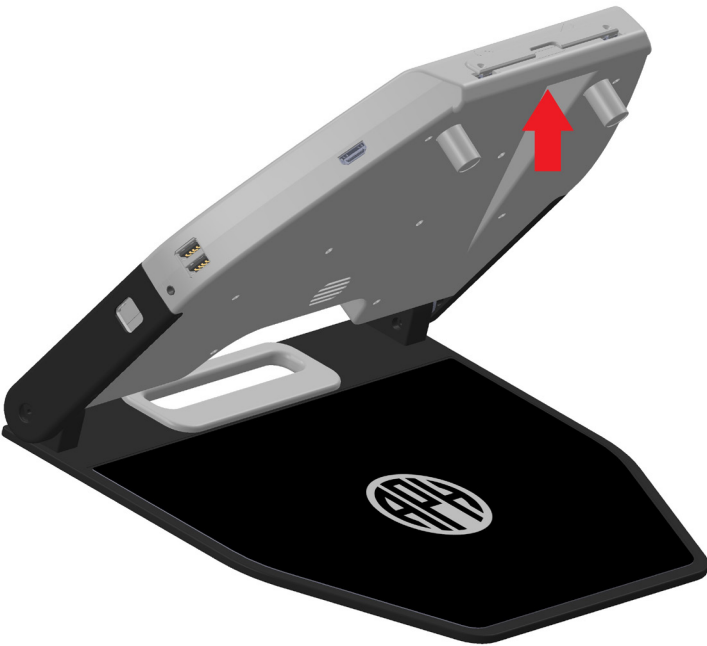


3. Attach the display screen: Insert the display screen's hooks into the holes at the top of the Neptune. The screen hooks will click into place.

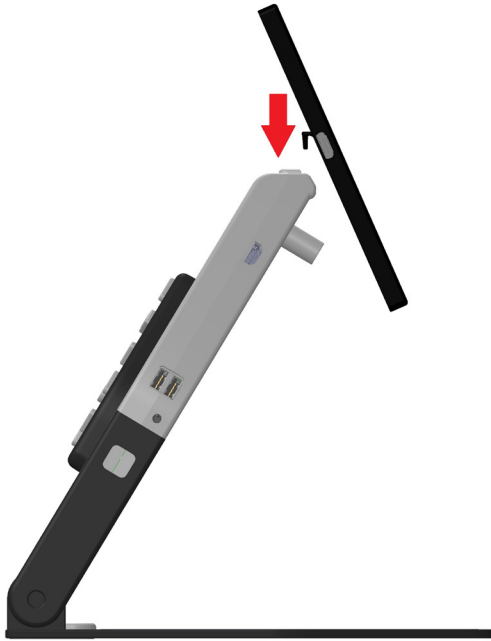

The display screen's hook holes are on the top surface of the Neptune.

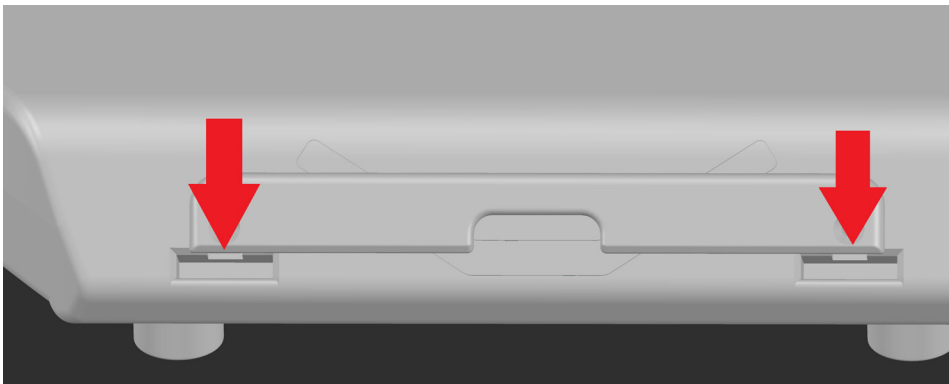

4. Use the enclosed, short, HDMI cable to connect the display screen to the Neptune.

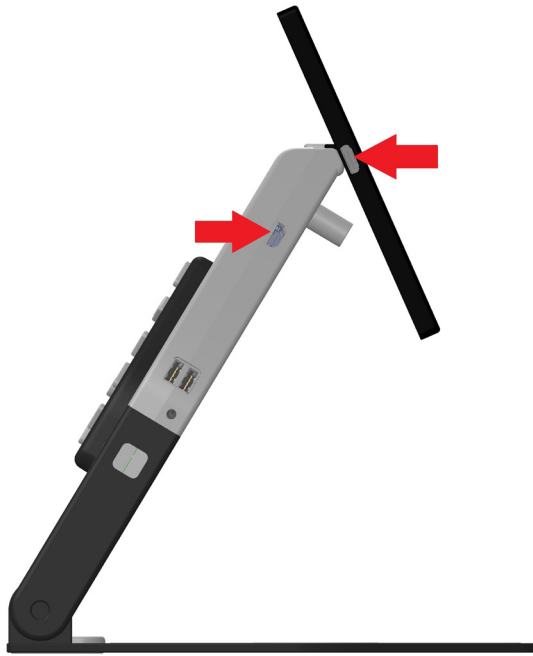


5. Attach the AC adapter at the right side of the Neptune, just above the round Power button, and plug into a wall outlet.
6. Press the round Power button on the right side of the Neptune.
7. Once you hear the series of beeps and the “Place your document” announcement, place your document, in landscape orientation, on the Neptune’s baseplate. You will hear a camera shutter sound, and the document will be read aloud and displayed on the Neptune’s display screen.

8. Use the Neptune's remote Keypad to adjust the voice, reading speed, and the size and color scheme of the text to best suit your preferences.



9. The user must press the Open/Close button on the back left side to release the arm, then close the device. If they push down too hard without pressing the button, they could break the device.

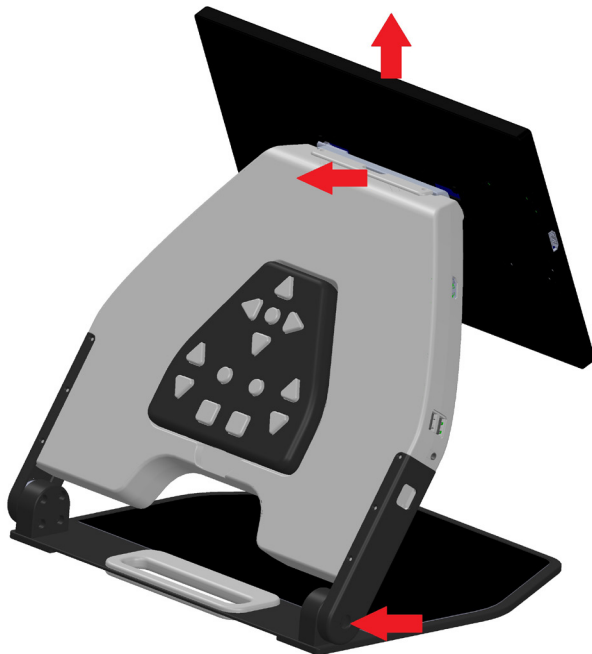

Neptune

Catalog Number: 1-00430-00

Copyright © 2025



**AMERICAN
PRINTING
HOUSE** 

**1839 Frankfort Avenue
Louisville, Kentucky 40206
502-895-2405 • 800-223-1839
aph.org • info@aph.org**